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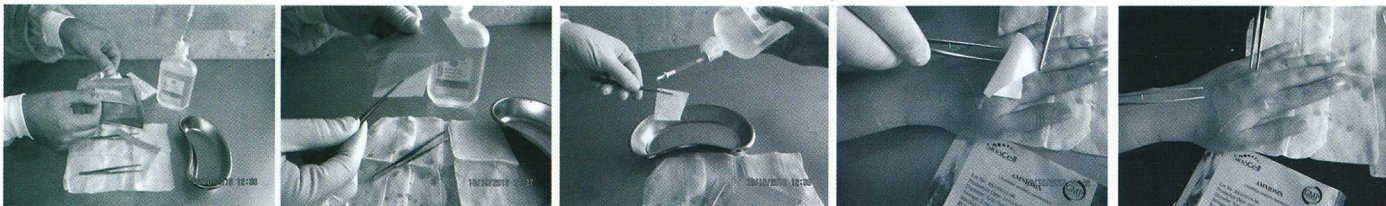
# Amniosin™

## ► Description

*Amniosin™* is a cryopreserved acellular human amniotic membrane-derived biological dressing. Due to the containing of different biological factors, *Amniosin™* has an important reconstructive role as a biological dressing.

The *Amniosin™* has been shown to provide two elements to the chronic wound repair process:

1. Supply of different bioactive extracellular matrix components such as collagen, Fibronectin, Laminin, Proteoglycan and glycosaminoglycan.
2. Function as a permissive, performed, normal human matrix for keratinocyte and fibroblast migration and proliferation.

## ► Indications and Clinical use

*Amniosin™* has been demonstrated in clinical trials to improve the healing of different wounds such as:

- Dressing and cover the burn wounds
- ophthalmic surgeries
- Diabetic ulcers
- Chronic wounds
- Urology surgeries
- ENT (Ears, Nose, Throat) surgeries

*Amniosin™* is indicated for use in the treatment of full-thickness diabetic foot ulcers greater than 6 weeks duration. *Amniosin™* should be used in conjunction with standard wound care regimens and in patients who have adequate blood supply to the involved wound.

## ► Contraindications

The presence of active infection or the ulcer extended to the tendon, muscle, joint capsule, or bone exposure.

## ► Precautions

- Do not reuses, refreeze, or sterilize the product.
- Do not use the product if there is evidence of container

damage.

- If the date and time stamped on the shipping box have expired.
- The product must remain frozen (at  $-75^{\circ}\text{C} \pm 5^{\circ}\text{C}$  or  $-20^{\circ}\text{C}$ ) continuously until ready for use.

## ► Adverse events

To date, no immunologic reactions to *Amniosin™* have been reported.

Dosage and administration

For the treatment wounds, the recommended treatment regimen is one piece of *Amniosin™* implanted every 5 days, until the ulcer is healed.

Before using *Amniosin™*, read full DIRECTIONS FOR USE.

## ► Directions for use

The ulcer must be surgically (sharply) debrided: removing any necrotic or hyperkeratinized tissue, leaving a wound bed that meets the clinical criteria for skin grafting prior to application of *Amniosin™*. If extensive bleeding is observed, the bleeding must be stopped before applying *Amniosin™*.

*Amniosin™* should not be used if infection is present in the wound. Clinical treatment should be initiated to resolve the infection. Once infection has cleared, treatment with *Amniosin™* can be initiated or resumed.

## ► Materials required for preparation and application:

- Sterilized scissors
- Surgical gloves
- Sterile normal saline (0.9% sodium chloride) at room temperature
- Sterilized forceps
- Dressing supplies
- Sterilized Gallipot

*Amniosin™* is packaged in 3 pockets. The innermost pocket is sterile and the outermost pocket which is included brochure is clean.

## ► Preparation for use:

1. Tear the First pocket and discharge the second pocket (which is included the *Amniosin™*) on the

sterile dressing pad.

2. Put the *Amniosin™* in the room temperature. Allow approximately 2-3 minutes for thawing until there are no ice crystals visible.
  3. Open the *Amniosin™* containing pocket with surgical gloves.
  4. Remove the *Amniosin™* from the pocket using sterilized forceps.
  5. Wash the *Amniosin™* with normal saline several times in a gallipot. (Shake it gently)
- NOTE: During the thawing and rinsing steps, do not touch the *Amniosin™*.
6. *Amniosin™* is on the nitrocellulose paper; Cover the surface of the whole wound by *Amniosin™* and pushes it gently on the wound to paste that on the wound. After that, pick out the nitrocellulose paper.
  7. If necessary, use of saline-moistened gauze as the second dressing supply.

## Important notes

1. Between routine applications of *Amniosin™*, it is important to maintain a moist wound environment.
2. If the second dressing is needed to change before 5 days, do not remove previously implanted *Amniosin™*.
3. After washing the *Amniosin™*, use it immediately to prevent tissue drying.
4. Remove the pocket label and put it on the patient chart. This label is included of Lot Number and expiration date.
5. Following each application of *Amniosin™*, the first signs of wound improvement will be observed in approximately first 5 days.
6. Dispose of all liquid, rinsing solutions, and *Amniosin™* components in accordance with institution or government environmental regulations.

## Storage recommendations

*Amniosin™* can be stored frozen for up to 6 months at  $-75^{\circ}\text{C} \pm 5^{\circ}\text{C}$  and up to 3 months at  $-20^{\circ}\text{C}$ . The insulated shipping container should contain dry ice (solid carbon dioxide) for distances which is more than 3 hours needed and for less than 3 hours distances, normal ice packs can be used.